



## CJD Fact Sheet

## Why are Transonic Neuro Flowprobes Single Use?

### References

- (<http://www.ninds.nih.gov/disorders/cjd/detail-cjd.html>)
- <sup>1</sup>Taylor DM, Fraser H, McConnell I, Brown DA, Brown KL, Lamza KA, Smith GR, Decontamination studies with the agents of bovine spongiform encephalopathy and scrapie. *Arch Virol* 1994;139(3-4):313-326.
- <sup>2</sup>Burger D, Gorham JR, Observation on the remarkable stability of transmissible mink encephalopathy virus. *Res Vet Sci* 1977 Jan;22(1):131-2.
- <sup>3</sup>Brown P, Liberski PP, Wolff, A, Gajdusek, DC, Resistance of scrapie infectivity to steam autoclaving after formaldehyde fixation and limited survival after ashing at 360 degrees C: practical and theoretical implications. *J Infect Dis* 1990 Mar;161(3):467-72.
- <sup>4</sup>el Hachimi KH, Chaunu MP, Cervenakova, L, Brown, P, Foncin, JF, Putative neurosurgical transmission of Creutzfeldt-Jakob disease with analysis of donor and recipient: agent strains. *C R Acad Sci III* 1997 Apr;320(4):319-28.
- <sup>5</sup>Manuelidis, L, "Decontamination of Creutzfeldt-Jakob Disease and Other Transmissible Agents," *Neuroviro* 1997;3(1):62-65.
- <sup>6</sup>Estebe JP, Anesthesia and non-conventional transmissible agents. *Ann Fr Anesth Reanim* 1997;16(8):955-63.
- Ghani AC, Ferguson NM, Donnelly, CA, Hagenaars, TJ, Anderson, RM, Epidemiological determinants of the pattern and magnitude of the vCJD epidemic in Great Britain. *Proc R Soc Lond B Biol Sci* 1998 Dec 22;265(1413):2443-52.
- Hill AF, Butterworth RJ, Joiner, S, Jackson, G, Rosser, MN, Frosh, A, Tolley, N, Bell, JE, Spencer, M, King, A, Al-Sarraj, S, Ironside, JW, Lantos, PL, Collinge, J, Investigation of variant Creutzfeldt-Jakob disease and other human prion diseases with tonsil biopsy samples. *Lancet* 1999 Jan 16;353(9148):183-9.
- Kirkwood JK, Cunningham AA, Wells, GA, Wilesmith, JW, Barnett, JE, Spongiform encephalopathy in a herd of greater kudu (*Tragelaphus strepsiceros*): epidemiological observations. *Vet Rec* 1993 Oct 9;133(15):360-4.
- Gibbs CJ Jr, Asher DM, et.al. Transmission of Creutzfeldt-Jakob disease to a chimpanzee by electrodes contaminated during neurosurgery. *J Neurol Neurosurg Psychiatry* 1994 Jun;57(6):757-8.
- Collins, S, Law, MG, Fletcher, A, Boyd, A, Kaldor, J, Masters, CL, Surgical treatment and risk of sporadic Creutzfeldt-Jakob disease: a case-control study. *Lancet* 1999;353(9154):693-7.
- "World Health Organization Consultation on Public Health Issues Related to Bovine Spongiform Encephalopathy and the Emergence of a New Variant of Creutzfeldt-Jakob Disease," *Morbidity and Mortality Weekly Report*, 12, April, 1996; 295-303.
- Manuelidis, EE, Manuelidis, L, "Suggested Links between Different Types of Dementias: Creutzfeldt-Jakob Disease, Alzheimer disease and Retroviral CNS Infections," *Alzheimer Dis Assoc Disord* 1989;3(1-2): 100-9.
- In the US, CJD is annually identified in the death certificates of 0.9 per million persons. Holman RC, Khan AS, Kent, J, Strine, TW, Schonberger, LB, Epidemiology of Creutzfeldt-Jakob disease in the United States, 1979-1990: analysis of national mortality data. *Neuroepidemiology* 1995;14(4):174-81.



**Transonic Systems Inc.**

Tel: 800-353-3569(USA); Fax 607- 257-7256; [www.transonic.com](http://www.transonic.com)  
Europe: Tel: 31 43 407 7200; e-mail: [info@transonic.nl](mailto:info@transonic.nl); Fax: 3143 407 7201

Copyright Transonic Systems, Inc. 1999.  
ML-36 CJDs Rev. B 3/05, Printed in the USA