# Charbel Micro-Flowprobe®

## Instructions for Use

# Micro-sonde de débit Charbel

## Mode d'emploi

# Charbel Mikro-Flußsonde

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# Charbel Micro-Flowprobe®

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### PRECAUTIONS

- The Flowprobe is for single use only.
- After its initial connection to the Flowmeter, the Flowprobe is functional for 16 hours. (Refer to the Operator's Manual for additional Warnings, Precautions and Features.)

1. Using aseptic technique, open the outer package and present the sterile contents to the scrub nurse to open and remove the Flowprobe. Pass the Probe connector to the circulating nurse who connects the Probe to the Flowmeter. The display message on the Flowmeter changes from "No-Pr" (no Probe connected) to "Ac-Er" (Acoustic Error) when a Probe is connected, but before there is ultrasonic contact with the vessel.

2. **Important:** Measure the diameter of the vessel to determine the appropriate size Micro-Flowprobe to be used. In some cases, more than one Probe size is needed to assess all the vessels of interest. Apply the Probe to the vessel of interest so that the vessel lies completely within the sensing window of the Probe, as shown. If the vessel extends outside this area, the total flow will be underestimated.

3. There is usually enough saline or cerebral spinal fluid in the field to promote good ultrasonic contact between the Probe and the vessel. If necessary, add sterile saline. Once good contact is established, the Flowmeter automatically displays mean volume flow in milliliters per minute.

4. The arrow on the Probe body indicates positive (forward) direction of flow. If the flow readings are negative, the direction of flow is opposite to the arrow indication.

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**AMERICAS** Transonic Systems Inc.
**EUROPE** Transonic Europe B.V.