

Technical Note: HD03 Sensor Placement on HD Bloodlines with Fresenius Twister

At the right is a diagram that shows proper placement of HD03 Flow/dilution Sensors to measure **Delivered Flow**, **Recirculation** and **Access Flow** with a Fresenius Twister and the following:

- 1) **HD03 software:** Version 2.2.2 or higher
 - 2) **Tubing:** 276 and higher
 - 3) **Tubing list:**
Select FRESENIUS MEDICAL CARE TWISTER F00000193
- * If using a Flow-QC Set to measure **Cardiac Output**, contact Tech Support for instructions.

Tech Support Hotline:

1-800-353-3569 (USA/Canada)

or 1-607-257-5300

