

Scisense 7F Catheter Usage Guide

Transonic Scisense 7F Pressure-Volume (PV) Catheters are intended for cardiac Pressure-Volume research. Transonic Scisense 7F Pressure & PV Catheters are intended for animal research models such as pig, sheep and cattle. The Catheters and associated electronics are NOT meant for human use of any kind.

The Catheter has been designed to provide several surgical options. The tip is configurable for various modes of insertion. The Catheter has a central lumen for either a 0.035" guidewire or injection port. A guidewire is included with the Catheter.

THE GUIDEWIRE IS AN INTEGRAL PART OF THE CATHETER STRUCTURE AND MUST BE IN PLACE WHILE THE CATHETER IS BEING INSERTED OR WITHDRAWN.

FAILURE TO USE THE GUIDEWIRE DURING INSERTION/WITHDRAWAL MAY RESULT IN THE OUTER SHEATH KINKING AND INVALIDATING THE WARRANTY.

Catheter Specifications

- Outer Sheath: Polyurethane tube reinforced with flat wound SS wire
- Inner Sheath: 0.040" ID polyimide
- Guide wire: 0.035" recommended & supplied
- Functional lengths 110 cm
- Tip: 20% barium filled pigtail 1.2 cm radius closed end
- Electrical connection: Scisense proprietary HDMI style connector
- Single Piezo resistive pressure sensor located 25 mm from distal end of Catheter

Accessories for Scisense 7F Pressure & PV Catheters



Supplied Guidewire (FA-70GW1): 0.035" X 150 cm long J tip guidewire





Supplied Hemostasis Valve with Cap (FA-70HV1)
Cap not shown.



10F-12F Introducer
(Recommended, not supplied)

7F Catheter Usage Guide Cont.

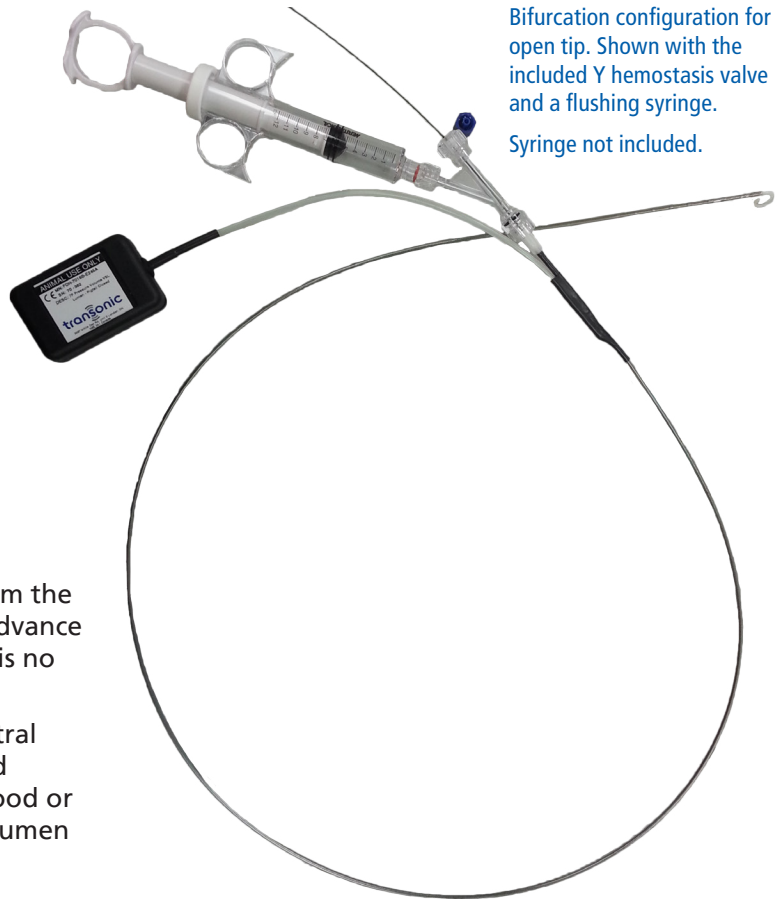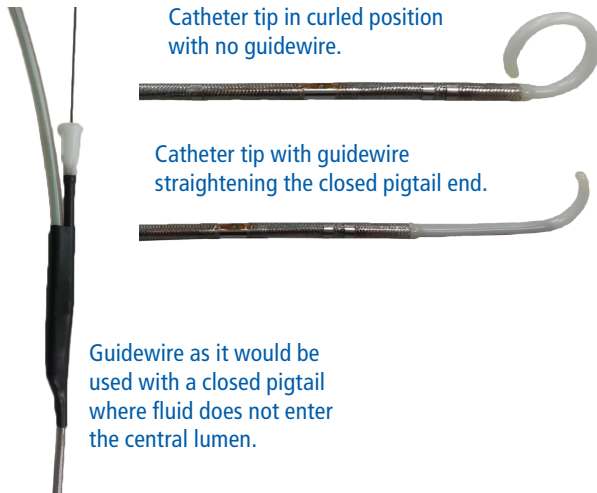

OPEN TIP OPTION

Cut the distal end of the curled pigtail 2 mm from the tip, the Catheter can be used to inject fluid or advance the guidewire past the tip. Once opened, there is no way to reseal the tip.

Opening the pigtail will allow fluid into the central lumen. In order to control the fluid, the supplied hemostasis valve must be used. Do not allow blood or any liquid to coagulate or persist in the central lumen after use.

CLEANING

See Scisense Catheter Cleaning Instructions. If the lumen has been opened at the tip, then the central lumen must also be cleaned with the appropriate enzymatic cleaner. For best results clean Catheters immediately after use.

STORING OF CATHETER

Transonic Scisense recommends that the Catheter be stored in the original package. Special care should be taken so as to not compress the pressure sensing element in the anti-static foam.



Transonic Systems Inc. is a global manufacturer of innovative biomedical measurement equipment. Founded in 1983, Transonic sells "gold standard" transit-time ultrasound flowmeters and monitors for surgical, hemodialysis, pediatric critical care, perfusion, interventional radiology and research applications. In addition, Transonic provides pressure and pressure volume systems, laser Doppler flowmeters and telemetry systems.

AMERICAS

Transonic Systems Inc.
34 Dutch Mill Rd
Ithaca, NY 14850
U.S.A.
Tel: +1 607-257-5300
Fax: +1 607-257-7256
support@transonic.com

EUROPE

Transonic Europe B.V.
Business Park Stein 205
6181 MB Elsloo
The Netherlands
Tel: +31 43-407-7200
Fax: +31 43-407-7201
europe@transonic.com

ASIA/PACIFIC

Transonic Asia Inc.
6F-3 No 5 Hangsiang Rd
Dayuan, Taoyuan County
33747 Taiwan, R.O.C.
Tel: +886 3399-5806
Fax: +886 3399-5805
support@transonicasia.com

JAPAN

Transonic Japan Inc.
KS Bldg 201, 735-4 Kita-Akitsu
Tokorozawa Saitama
359-0038 Japan
Tel: +81 04-2946-8541
Fax: +81 04-2946-8542
info@transonic.jp

